

SMART-FOLD* STERILIZATION WRAP

WRAPPING TECHNIQUE

The easier and faster way to wrap. **59% Faster.**¹



1

Place the front edge of the tray at the reference line and centre on top of reinforcement zones.



2

Take the first fold over the tray and cover it completely.



3

Gather side.



4

Fold side up and secure. Repeat steps 3 and 4 on other side.



5

Gather top at both sides and fold inward.



7

Secure as usual.



6

Fold over to cover tray.

For more information contact your Halyard Account Manager or please visit:
www.halyardhealth.com.au

1. Comparison of KIMGUARD ONE-STEP* Wrap to KIMGUARD* SMART-FOLD* Wrap: Kurt Salmon & Associates, Time and Motion Studies on the Labor Requirements of the Kimguard Smart-Fold, One-Step, and Sequential Sterilization Wraps, June 2011.

2. Halyard product specifications.

*Registered Trademark or Trademark of Halyard Health, Inc. or its affiliates. © 2015 HYH. All rights reserved. SS:SW:1505:WFHY637

Keep Green in Mind

SMART-FOLD* Sterilization Wrap is made from recyclable #5 polypropylene.



Use Once.
Recycle Always.

SMART-FOLD* STERILIZATION WRAP

UNWRAPPING TECHNIQUE

The easier and faster way to unwrap. **55% Faster.**¹
And inspect. **86% Faster.**¹



1

Break closure tape.



2

Unfold first layer.
Open sides
simultaneously.



3

Gently lift the tabs and
pull towards you.

For more information contact your
Halyard Account Manager or please visit:
www.halyardhealth.com.au

1. Comparison of KIMGUARD ONE-STEP* Wrap to KIMGUARD* SMART-FOLD* Wrap: Kurt Salmon & Associates, Time and Motion Studies on the Labor Requirements of the Kimguard Smart-Fold, One-Step, and Sequential Sterilization Wraps, June 2011.

2. Halyard product specifications.

*Registered Trademark or Trademark of Halyard Health, Inc. or its affiliates. © 2015 HYH. All rights reserved.
SS.SW.1505:WFHY637

Keep Green in Mind

SMART-FOLD* Sterilization
Wrap is made from recyclable
#5 polypropylene.



Use Once.
Recycle Always.